

Liqui-PREP™ SPECIAL PROCESSING PROCEDURE




STEP 1

Add cleaning solution to the labeled centrifuge tube.




STEP 2

Mix each specimen well.

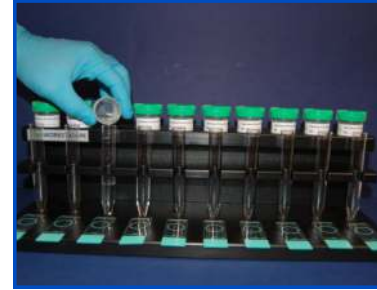


STEP 3

Pour specimen into the centrifuge tube.

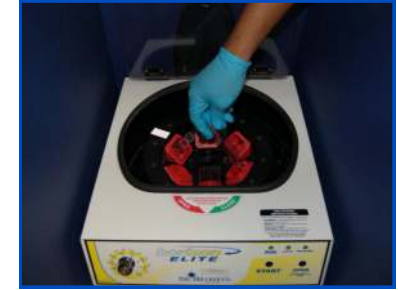


STEP 4

Centrifuge for 10 minutes @ 1000xg.

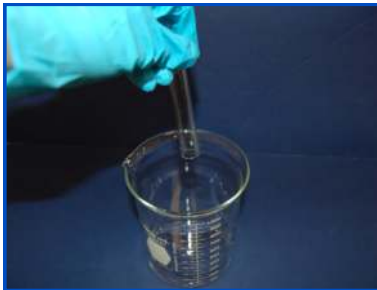

STEP 5

Pour off supernatant.

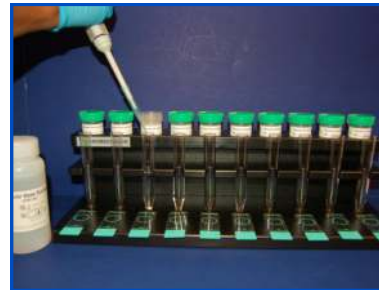


STEP 6

Dilute pellet with cell base.



STEP 7

Mix each specimen immediately before inoculation of slide.

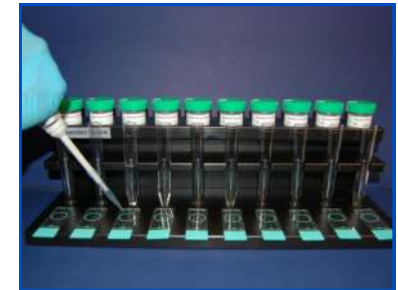

STEP 8

Make the slide, dry stain and read.